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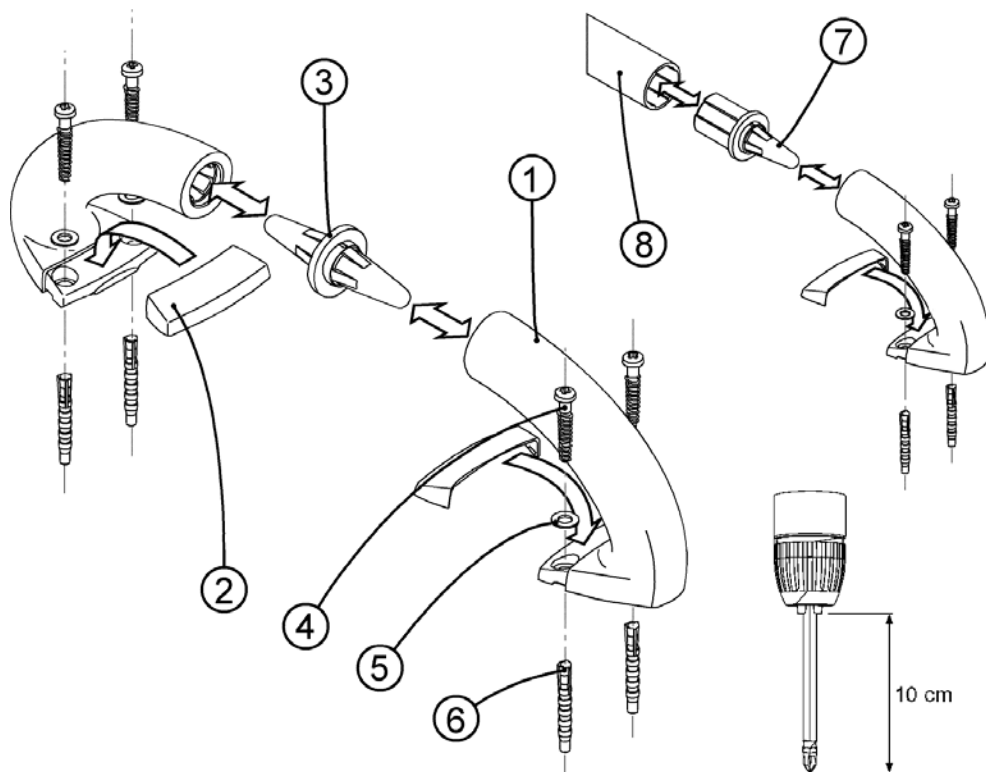
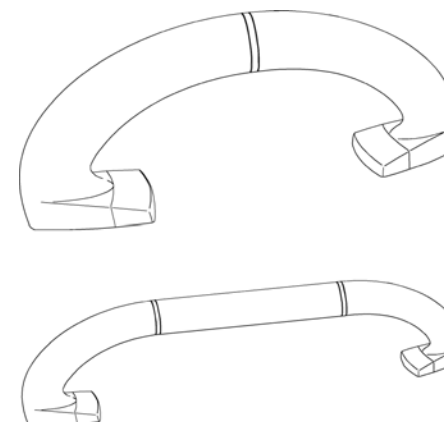
Handgrip 7760 (GB)

The 7760 is a handgrip available in various lengths, from 30 cm to 100 cm long.

It is destined for use as a general support element, which is also suitable for high humidity environments.

General Information

At Invacare we are proud that our customers have chosen for a high quality bath-, toilet- and shower support products.

Through years of accumulated experience in the development and production of these quality products, Invacare has firmly established its leading position in this market.

The 7760 fulfils the CE-safety standards, which are in conformity with the regulations for medical supports from June 14 1993 no. 93/42 EEG.

Guarantee

Invacare provides a full 2 year guarantee for the handgrips. Wear and tear from normal usage and negligence are not covered by the guarantee.

Invacare is not liable for personal injury caused by improper use of the hand grips.

General safety

Please make sure that the 7760 is assembled and installed according to the given illustrations and instructions.

Have a capable professional assemble and install the handgrip.

Use preferably the attachment materials delivered by Invacare when assembling and installing.

The attachment set delivered with the 7760 includes four screws, four plugs and four washers; they are only appropriate for attachment onto stony walls. For walls of cellular concrete or other similar materials, special attachment materials must be used.

Always use screwdriver bits of approx. 10 cm length, to avoid damaging the grip half.

The handgrip can be placed in any position against the wall. A common position of the 30 cm handgrip is shown in the adjoining figure.

If you have any questions or in the case of any defects you always have to contact Invacare.

Important

When attaching the handgrip to the wall, the RVS washers (# 5 of the part list) must always be placed in their destined openings of the griphalf.

Maintenance

After use, the 7760 can be cleaned by water and normal household cleansing products. Avoid using abrasive scrubbing materials or strong solvents.

Technical information

- The handgrip is manufactured from solid, impact-proof ABS.
- Aluminum tube is coated with epoxy.
- The 7760 is available in lengths from 30 cm to 100 cm.
- Maximum weight permitted : 100 kg in all positions.

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