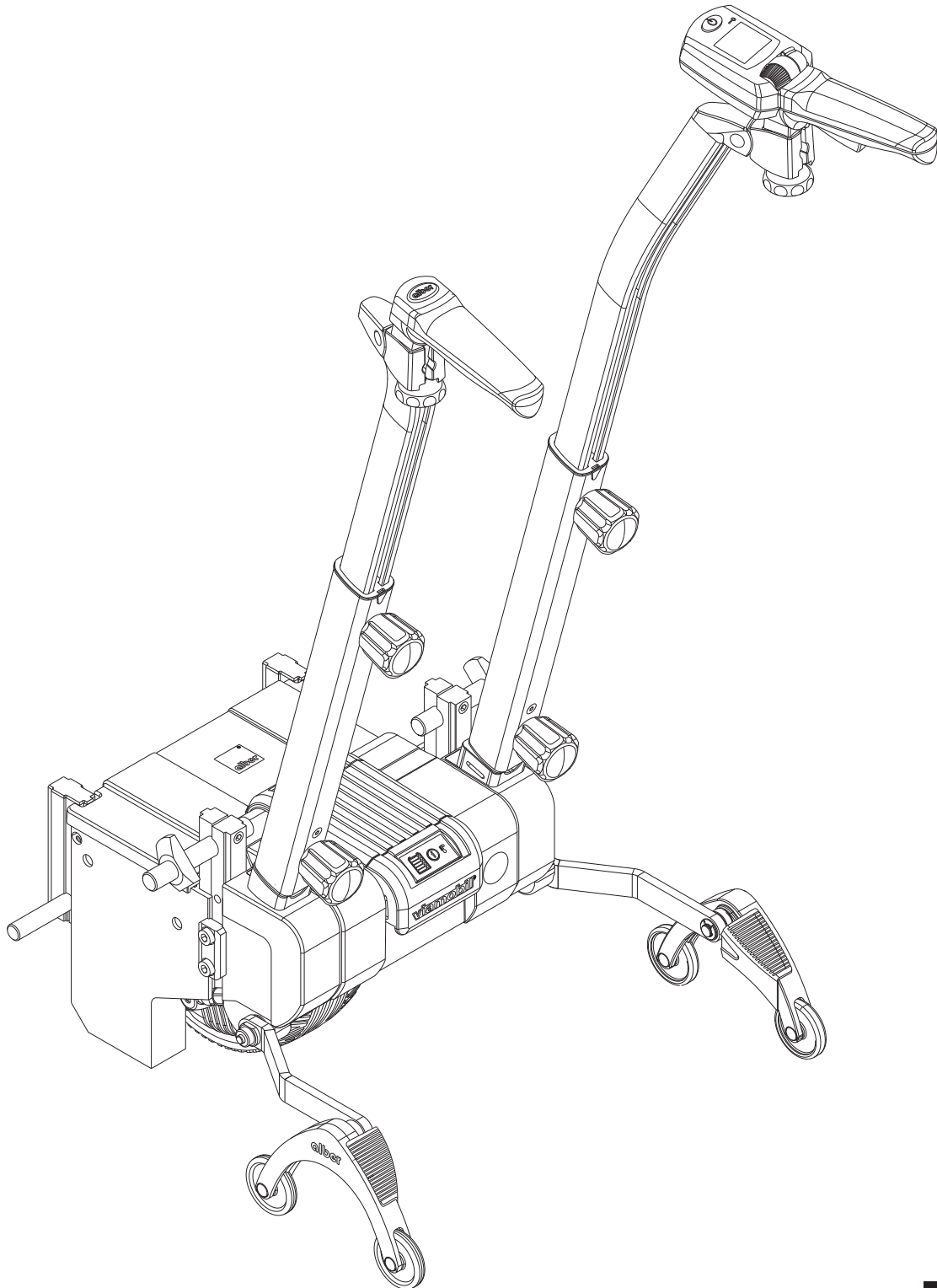
viamobil V25[®]

Anbauanweisung
Mounting instruction
Instructions de montage

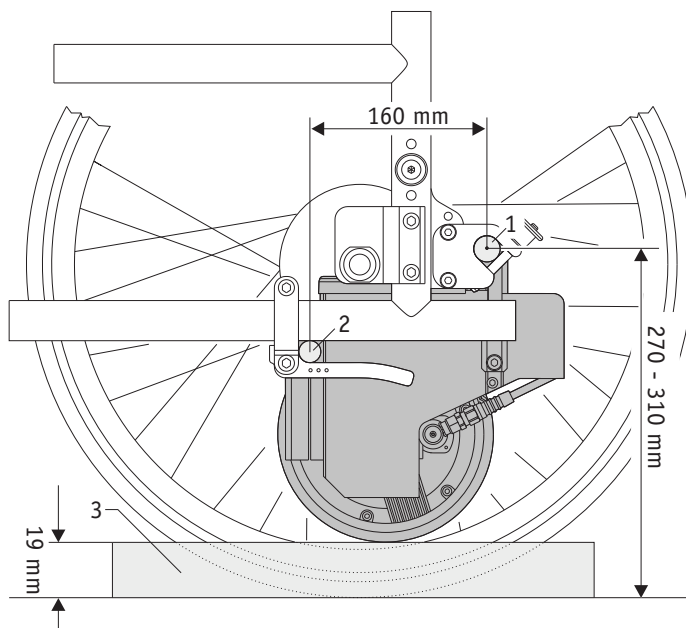
Hinweis zur Anpassung des viamobil an den Rollstuhl
Information about the Mounting of the viamobil at the wheelchair
Instructions de montage du viamobil sur le fauteuil roulant

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Mount the bracket on the wheelchair before you begin the installation work on the viamobil. Please take care of the distances indicated on the drawing of the upper [1] and lower bolts [2] which apply to most manual wheelchairs. The values might differ from the graph depending on the wheelchair model.

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
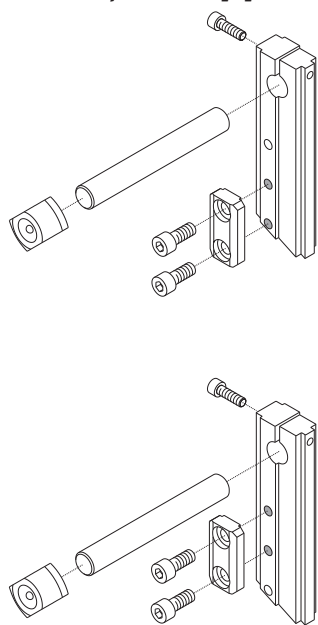
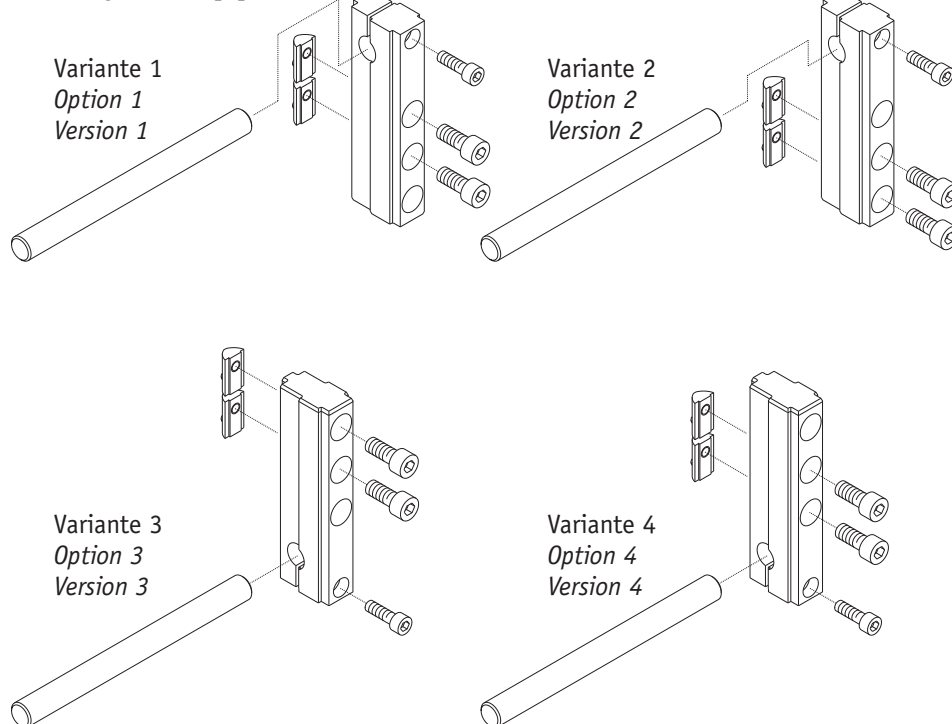
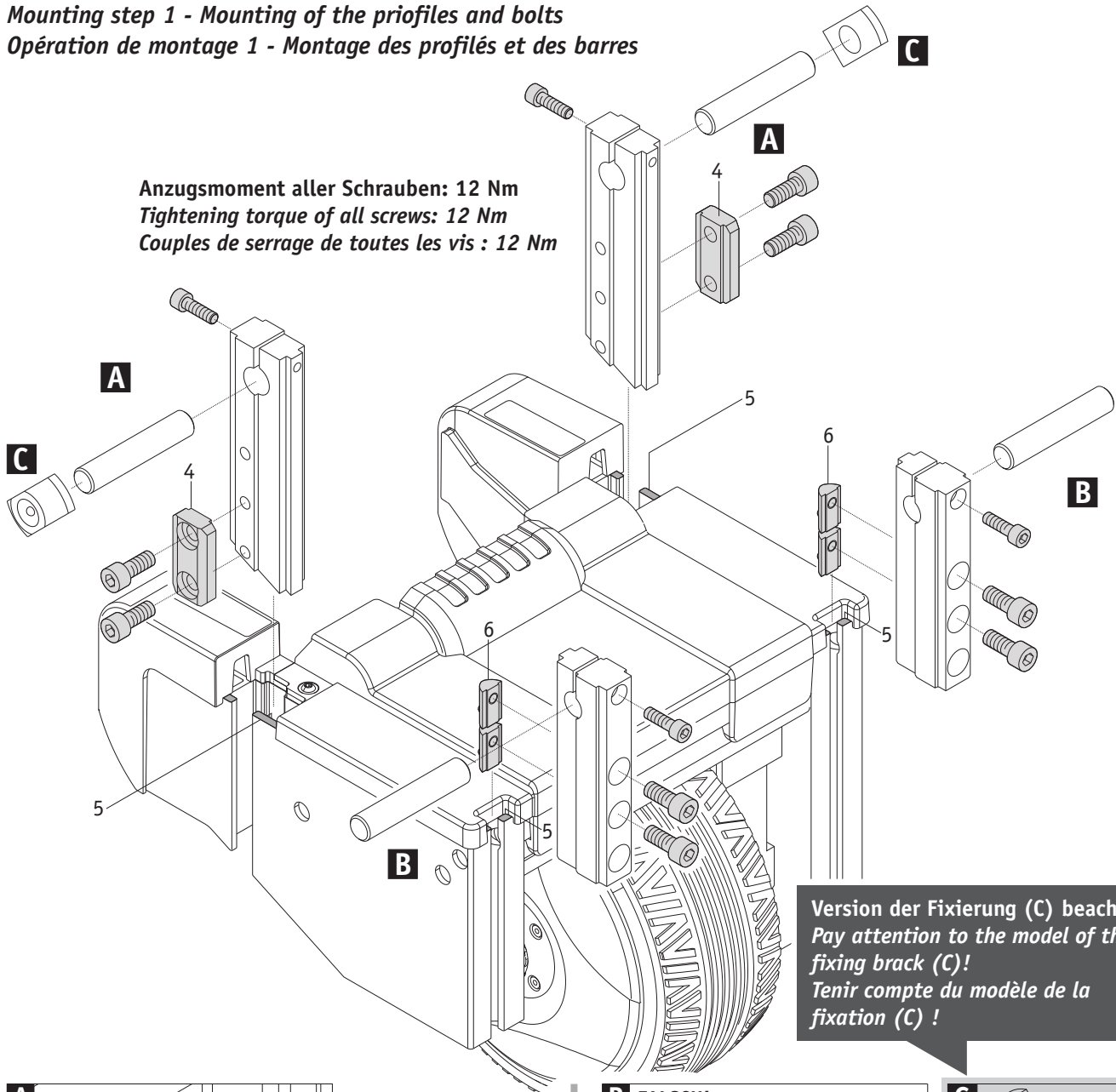
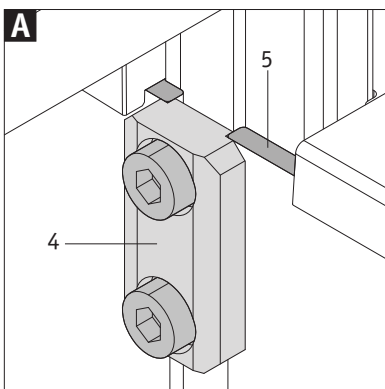
The counterpart [4] shall not exceed the edge of the guide rod [5] on the drive unit, but must completely touch the housing of the viamobil.

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


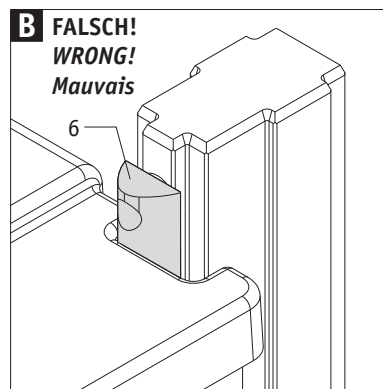
The screw nut [6] shall not exceed the edge of the guide rod [5] on the drive unit, but must stay completely within the guide rod [5].

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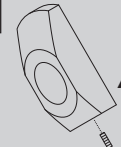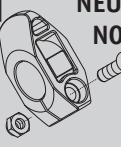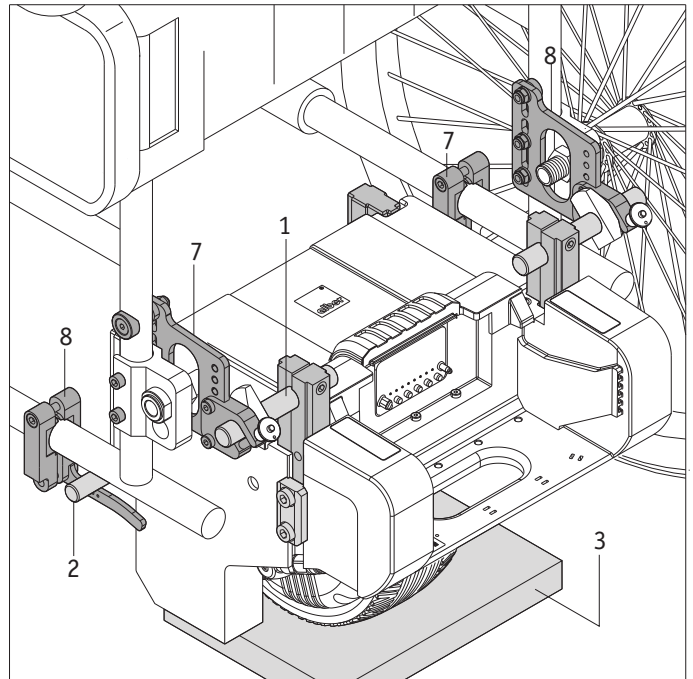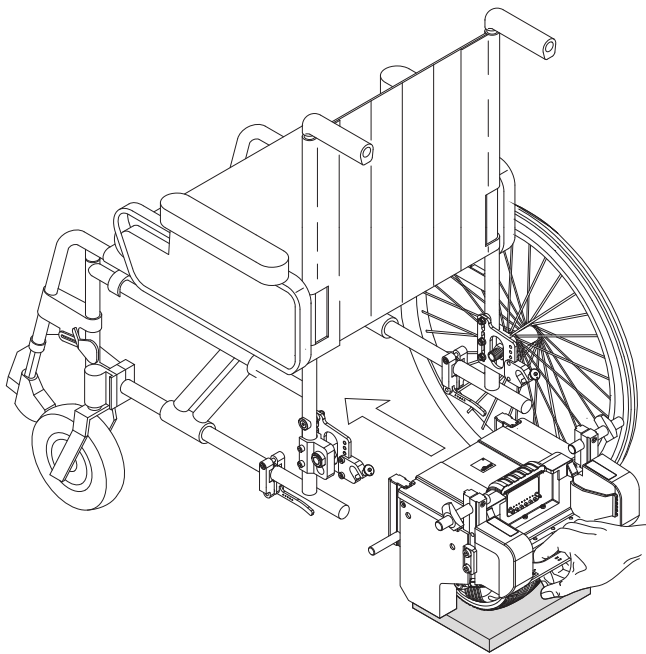Pay attention to the following instructions when mounting the viamobil:

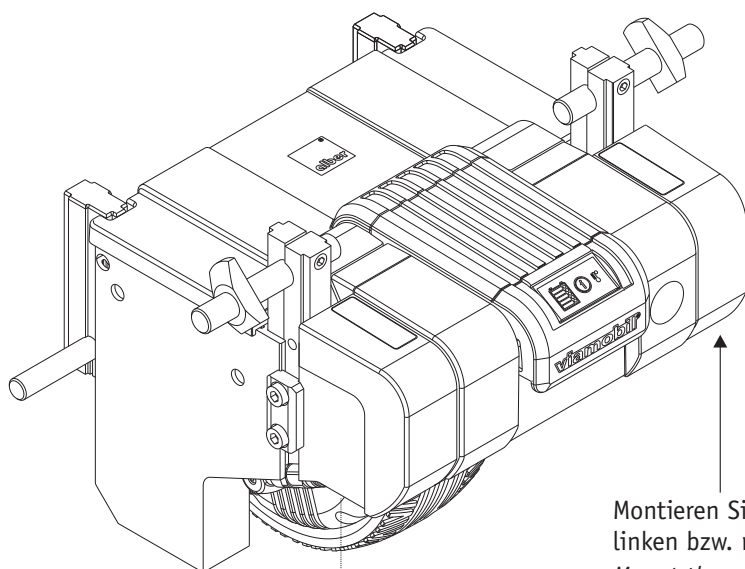
- The manual wheels of the wheelchair need to be fixed on the chair to adjust the viamobil. Please check the air pressure of the tyres.
- While adapting the viamobil to the chair, the profiles should still be height adjustable so the screws should not be tightened too much.
- Push the drive unit underneath the wheelchair. viamobil has to be switched off, thus the drive wheel must not be pulled out.
- Insert the lower bolts in the respective part of the mounting bracket [8].
- Insert the upper bolts in the fork of the upper mounting bracket [7] and fasten them by using the Quickpins.
- Put a 19 mm thick plate [3] (e. g. made of wood or steel) under the viamobil drive wheel. Alternatively, you can use the Alber mounting aid (part no. 1521404).
- Align the viamobil drive unit using a spirit level.
- Fix the bolt receiving profiles in this position. Tighten the screws to a torque of 12 Nm.
- Remove the plate [3]. The drive wheel decompresses a little bit. The distance from the wheel to the ground must now be 15 mm to ensure safe traction when pushing and braking the wheelchair.

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

Mount the anti-tippers in the outside notch at the left respectively right side of the housing.

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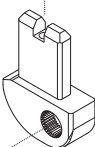
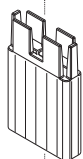
1. Add all parts and screw the screw (no more than three turns).
2. Insert the parts in the housing of the viamobil.
3. Tighten the screw, tightening torque: 4 Nm

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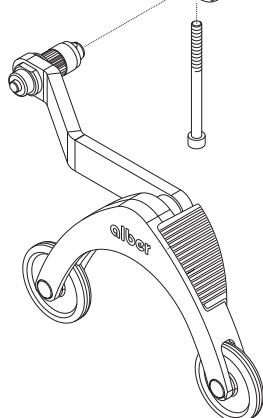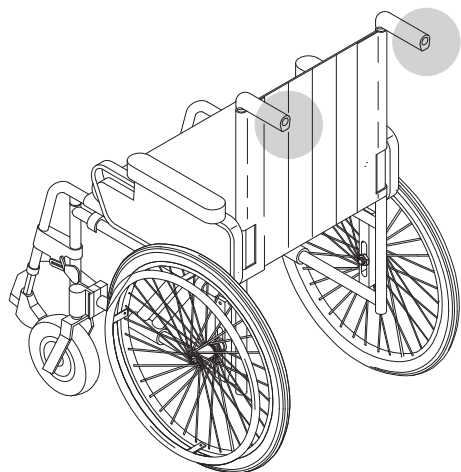
[A], [B] and [C]:
 The tightening torque depends on
 the material and the thickness of
 the tubes of the wheelchair.

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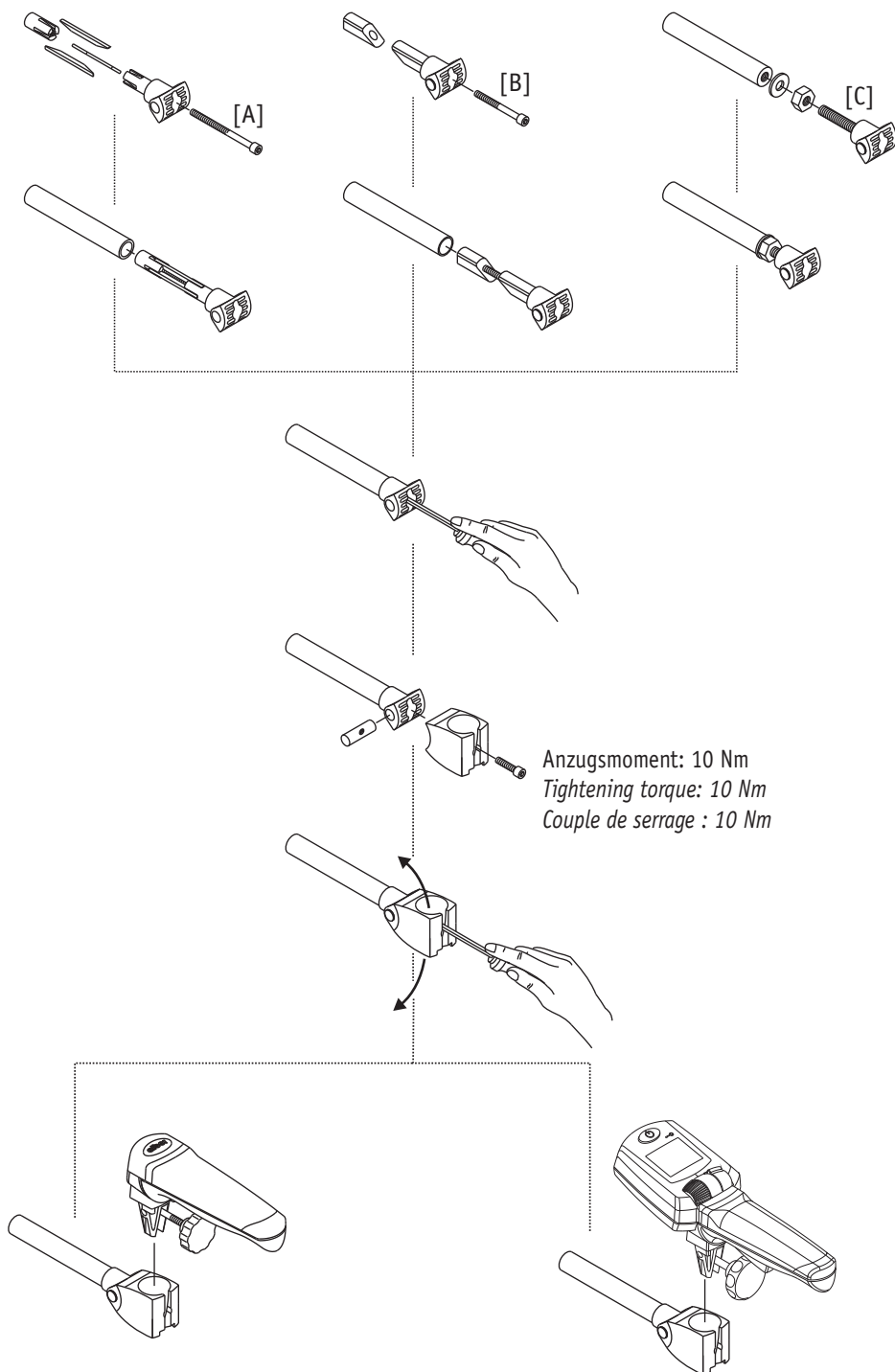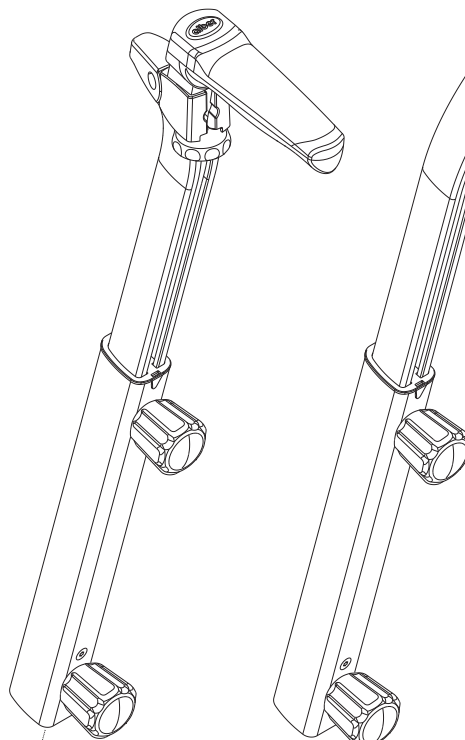
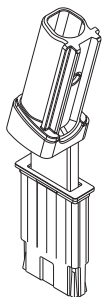
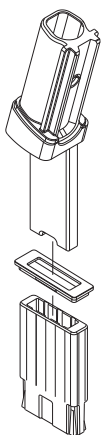
*First mount the viamobil at the wheelchair.
Then mount the additional handgrips at the viamobil.*

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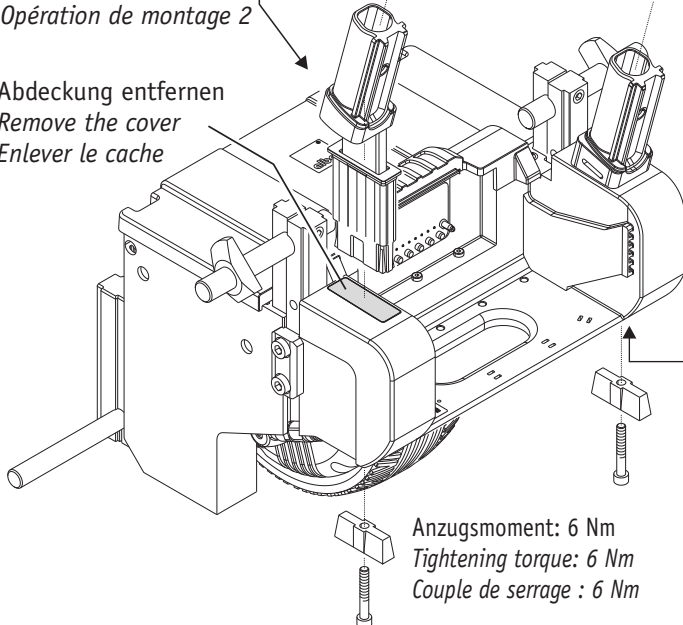
Mount the additional handgrips in the interior notch at the left respectively right side of the housing.

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Alber GmbH