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EN The information on the attached pages are meant for authorized Invacare® dealers only and are therefore not available in all languages.

At Invacare, our number one priority is to give you products that meet a wide variety of customer needs while maintaining their safety. The Bora/Spectra XTR continues to be highly popular because it is one of our most compact, lightweight, and transportable products available with a wide array of seating and positioning options. The new Modulite™ system has increased this flexibility with the addition of easy depth and center of gravity adjustment. With this, we have found that in a few of the more extreme combinations of seating system settings and customer weight distributions, the system can be less stable rearward increasing the likelihood that it will tip onto the anti-tip wheels. As a result, each time you receive a Bora with a recline, it will come pre-installed with a safety adapter to ensure each and every client is safe. In the majority of cases this can be removed and the full 120 degrees of recline can be utilized. The following details the procedure that can be used to verify that it is indeed safe to remove for your individual customer. These steps should only be performed by an authorized Invacare® dealer.

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