


Kompas UM Addendum

Armrest Width Adjustment

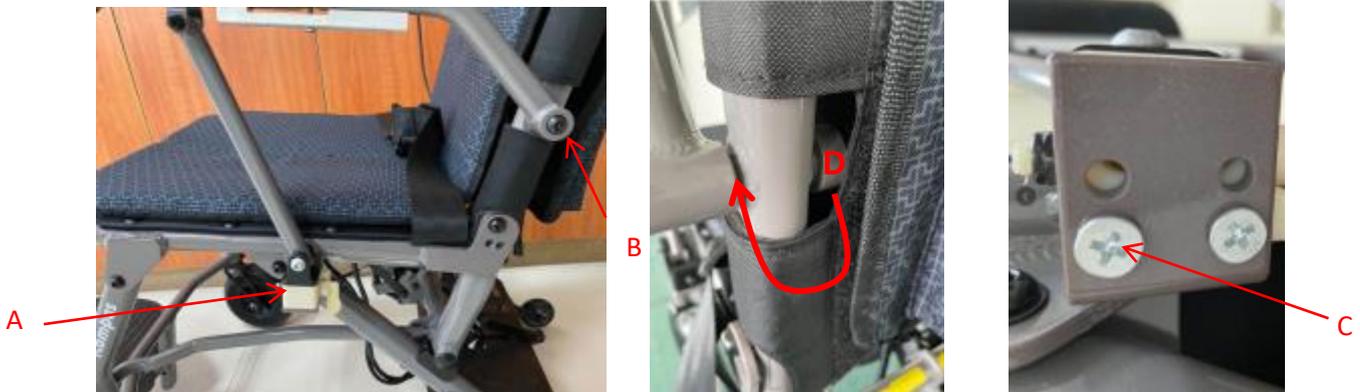

1. Unlock (A) to lift the armrest.
2. Remove the 2 screws(C) to the other two holes
3. Unscrew (B) and move the washer (D) from inside of the frame to outside and put it between armrest and the frame. Screw into the armrest fixation points (B) through the washer (D).
4. Place Armrest back to fixation points (A).
5. Repeat 1-5 on the other side.

Armrest Height Adjustment





1. Remove the screws from the armrest.
2. Remove (A) and relocate it between armrest and tube.
3. Lock screws on the armrest.
4. Repeat 1-3 on the other side.

Armrest Position Adjustment



1. Move one screw from 2nd to the 1st hole.
2. Repeat Step 1 on the other side.

Detachable armrest

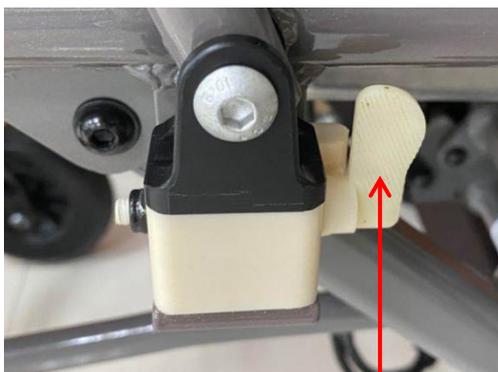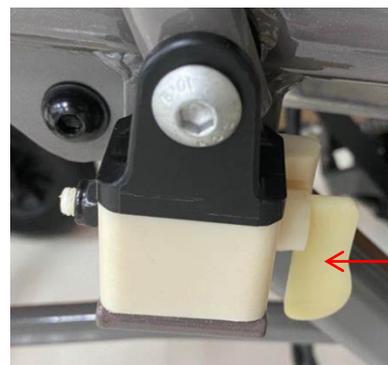

A

1. Rotate the Lever (A) 180 degree to unlock to detach.
2. Repeat 1 on the opposite side.

Backrest Adjustment



1. Remove the screws within the 'Fixing Holes' on both sides of the backrest.
2. Push back the backrest by 5 degrees.
3. Gently fix screws within the Fixing Holes on one side of the backrest and then proceed to do the same on the other side. Once gently fixed, proceed to tighten the screws on both sides.
4. To move the backrest forward, repeat Step 1 to 3 but fix the screws in reverse positions. .