Rea® Dahlia® Foldable / Pliable

Dahlia 30, Dahlia 40

en **Folding mechanism**
Service Manual 1

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I General

I.1 About this manual

This manual is an attachment to the user manual of the related product and it belongs to a special functionality. In order to ensure safety when using the product, read the user manual of the related product carefully and follow the safety instructions.

In this user manual, warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

I.2 Folding / unfolding the wheelchair

The wheelchair can be folded by removing the quick release pin of the seat tilt mechanism.

The backrest of the wheelchair can be folded as well. See user manual of the wheelchair for further instructions.

Removing the quick release pin

CAUTION! Risk of injury


– Do not sit on the wheelchair while it is folded.

IMPORTANT!

– Before removing the quick release pin, adjust the seat tilt to the most upright position.
– See user manual of the wheelchair for more information about tilt adjustment.

1. Turn the wheelchair on its back, so that it lays on the push handles and the rear wheels. Locate the quick release pin (A).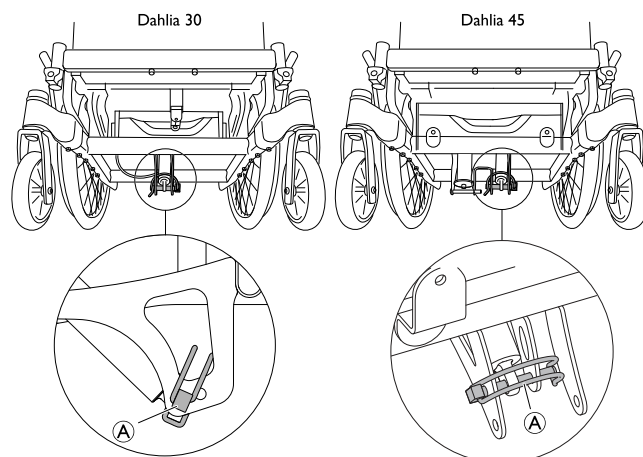


IMPORTANT!

At the Dahlia 45 there are two gas pistons for the tilt mechanism. The quick release pin is located at the thinner gas piston.

– Do not remove the fixed pin of the thicker, spring loaded gas piston.

2. Hold one hand below the the gas piston (B) and the attachment. With the other hand open the safety shackle (C) to unlock the quick release pin (A).

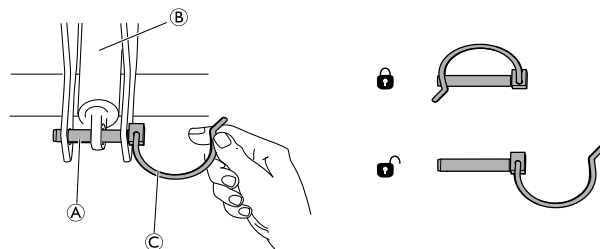
DEALER: Keep this manual.
The procedures in this manual MUST be performed by a qualified technician.


Yes, you can.®

3. Keep the safety shackle © open , pull the quick release pin (A) out of the brackets and let the and gas piston fall into your hand.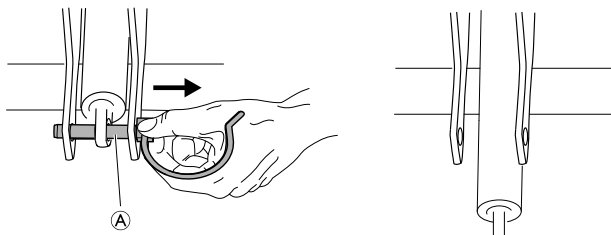

The seat tilt mechanism is now inactive.

4. In order to not lose the quick release pin, re-insert it in the brackets (without the gas piston).

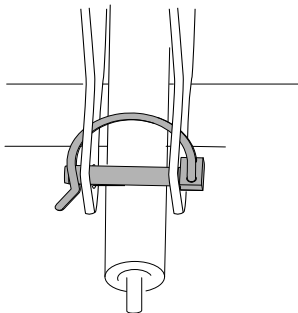Inserting the quick release pin


CAUTION!

Risk of injury due to rapid change of seat position

When the quick release pin is not inserted correctly, it might fall out of its position. The seat unit would then tilt unexpectedly forward or backwards.


- Make sure that the quick release pin is inserted correctly and locked in its position.

1. Turn the wheelchair on its back, so that it lays on the push handles and the rear wheels. Locate the quick release pin (A).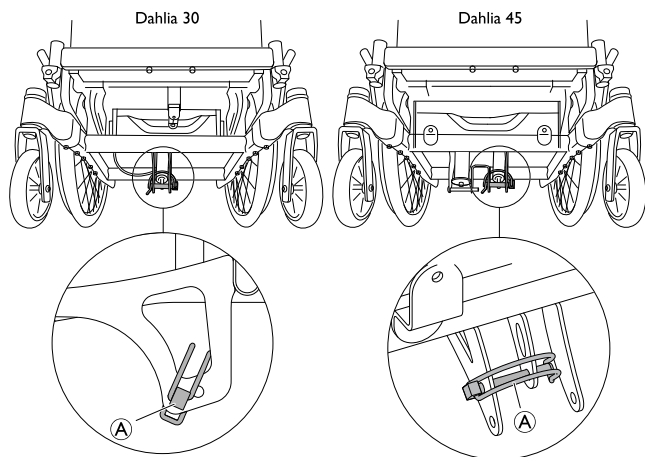

2. Open the safety shackle and remove the quick release pin from the brackets. Keep the quick release pin in your hand, unlocked.

3. Hold the gas piston (B) with the other hand and position it between the brackets (D), so all holes are in a line.

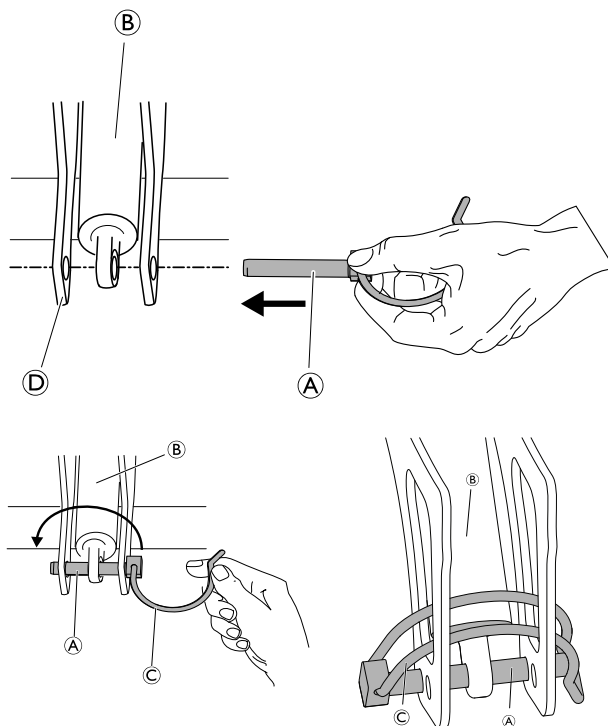4. While it is unlocked, push the quick release pin (A) through the holes of the brackets (D) and the gas piston (B).
5. Fold the safety shackle (C) over the quick release pin (A) to lock it. Make sure that the quick release pin is inserted correctly and locked and it can not be pulled out of its position.

The seat tilt mechanism is now active and can be adjusted.

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All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Invacare Poirier SAS
Invacare Rea AB