Invacare® H440 Omega Eco

- EN** **Commode chair**
User Manual
- DE** **Toilettenstuhl**
Gebrauchsanweisung
- FR** **Chaise toilettes**
Mode d'emploi
- IT** **Sedia da comodo**
Istruzioni per l'uso
- ES** **Silla WC**
Manual de instrucciones
- NL** **Toiletstoel**
Gebruiksaanwijzing
- DA** **Toiletstol**
Brugsanvisning
- FI** **WC-tuoli**
Käyttöohje
- NO** **Toalettstol**
Brukerveiledning
- SV** **Toalettstol**
Bruksanvisning
- PT** **Cadeira de WC**
Manual de instruções

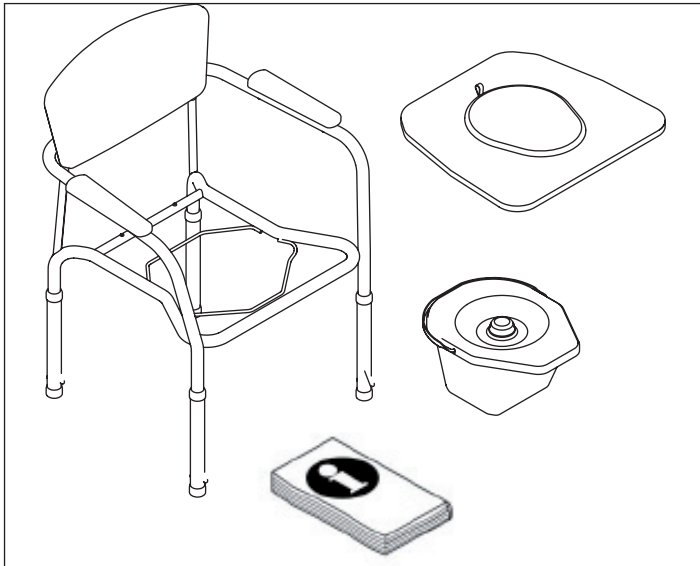
Fig. 1

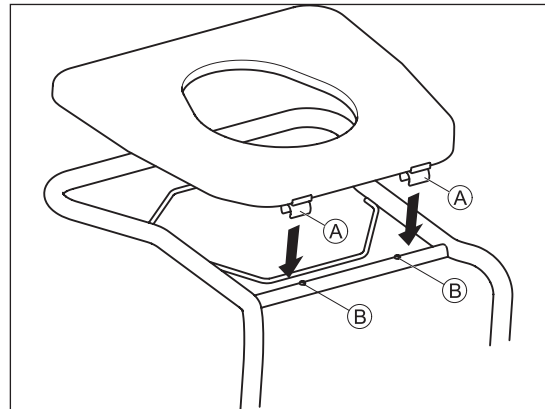
Fig. 2

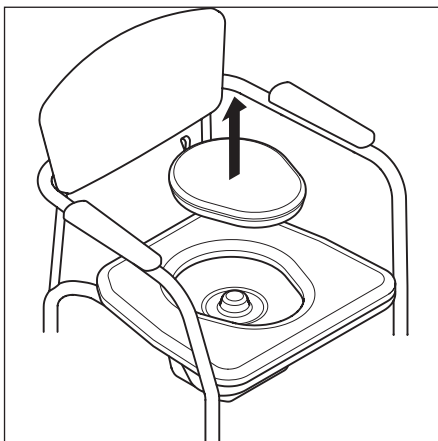
Fig. 3

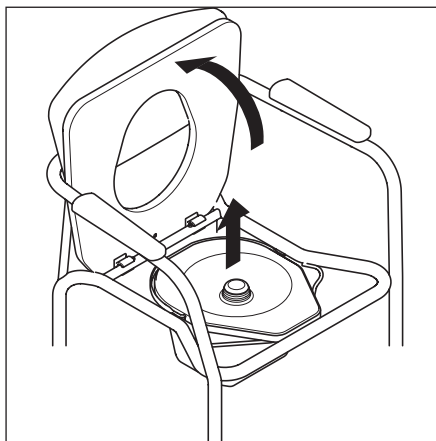
Fig. 4

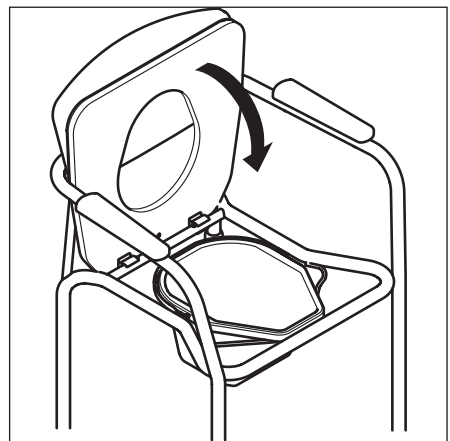
Fig. 5

INVACARE AQUATEC Operations GmbH
Alemannenstr. 10
D-88316 Isny

H440 Omega Eco

2012-02 140 kg

4 028698 114478

CE

LN (01)04028698114478(10)12BFS0001

INVACARE logo and icons for user manual and safety.

This manual must be given to the user of the product. Before using the product, read this manual and save it for future reference!

Yes, you can.

Intended Use

The commode chair is to be used solely as an aid for going to the toilet. Any other use is prohibited.

WARNING Risk of falling!

- Do not stand on this product.
- Do not use as a climbing aid.

Temperature for use: 10 - 40 °C

Max. user weight: 140 kg

Service Life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

Safety

WARNING Risk of injury!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Protect the product against exposure to heat.

Scope of delivery

- Fig. 1.

Assembly

CAUTION Risk of pinching!

- Take care during assembly.

WARNING Risk of falling!

- Ensure that all legs are adjusted to the same height.
- Ensure that the pins are correctly engaged after adjustment.

- Fig. 2.

WARNING Risk of falling!

- Ensure that, externally in each case, the clips Ⓐ engage next to the screws Ⓑ on the seat plate.

Usage

IMPORTANT

- Do not use the product in the shower.
- Use the product only at room temperature.

- Fig. 3 - 5.

Cleaning and disinfection

The product can be cleaned and disinfected using commercially available agents.

- Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

IMPORTANT

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Healthcare Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG002 and your local infection control policy.

- Clean by hand.
- Clean without scouring agent.

Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined above are adhered to.

WARNING

- Check product regularly for damage and to ensure that it is securely assembled.

Re-Use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly and check the safety functions, and reset if necessary.

Disposal

The disposal and recycling of used devices and packaging must comply with the applicable legal regulations.

Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business. Guarantee claims can only be made through the relevant specialist dealer.

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Invacare A/S, Sdr. Ringvej 37,
Invacare AB, Fagerstagatan 9,